



denovodental.com

ISO
ISO
Certified

CE
CE
Certified

FDA
FDA
Registered

Autoclavable
Autoclavable

Surgical
Stainless Steel

★
Made in
the USA



Stainless Steel Crowns

Application Procedures



1.800.854.7949



1.800.847.8599



info@denovodental.com




Responsibly Printed
& Packaged

Made and assembled in the USA
DENOVO Dental Inc. Baldwin Park, CA 91706




Crown Application

1

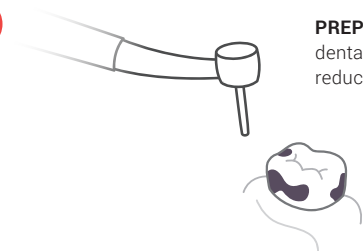

PREP: Prep the tooth for crown placement according to dental industry standards, practices and preferred reduction method.

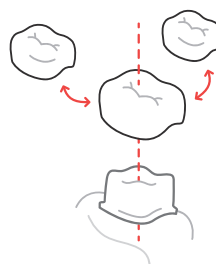

TRIAL FIT: Trial fit to select the proper crown size.
If trial fit is incorrect, **sterilize the crown for future use***.

2



TRIM: Trim the crown to the proper height with curved, stainless steel cutting scissors, making sure to match the contour of the gingival margin. Crown height should match the original tooth before prep, whilst the bottom edge of the crown should not be more than 1mm below the gingival margin to prevent irritation or inflammation.

CONTOUR: Contour the crown walls using crown crimping pliers and bend the gingival third of the crown inward to achieve a tight marginal fit and smooth marginal outline. There should be no or minimal gingival blanching.

3



SEAT & CEMENT: Seat the customized crown with your preferred glass ionomer and check occlusion.

*Crowns can be sterilized using autoclave, chemiclave or dry heat.

Have additional questions? Need to re-order? Call us at:

1.800.854.7949

FREE Ground Shipping on Orders Over \$300

Continental US only, certain restrictions apply, see website for full details



denovodental.com



CE Certified
2195



Denovo Dental Inc.
5130 Commerce Dr.
Baldwin Park, CA, USA



KinderDent, Gutenbergstrasse 7
D - 28844 Weyhe, Germany



Denovo Bands are made from surgical grade stainless steel, which contains nickel (9.58%). Nickel has been shown to cause allergic reactions in some patients.



Do not reuse. Appliances are designed and priced to be disposable. If appliance is used for a trial fit only, sterilize for future use.



Aspiration/Ingestion possibility. Use rubber dam on installation and educate patient to avoid hard or chewy substances which could dislodge appliance.

Denovo products are sold clean but not sterile; sterilization prior to use is recommended for patients with compromised immune systems.

REV 11