

# DENOVO<sup>®</sup>



denovodental.com

ISO  
Certified

FDA  
Registered

Autoclavable

Surgical  
Stainless Steel

★  
Made in  
the USA



# Strip-T Matrix System

## Application Procedures



1.800.854.7949



info@denovodental.com




Responsibly Printed  
& Packaged

Made and assembled in the USA  
DENOVO Dental Inc. Baldwin Park, CA 91706



# Strip-T Application


1

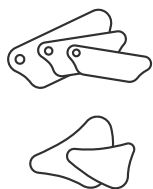

Straight Strip-T matrices are optimal for class II Molar or anterior restorations. Multiple sizes provide different heights to match the tooth and depth of the caries. Dental floss can be threaded through the hole to facilitate removal and promote safety.

Curved anterior matrices can be used for class III, IV or V restorations, and for facial veneering.

2

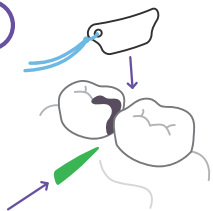


Slide the chosen Strip-T matrix between teeth. Use the provided woody wedge, or double-up the matrices to firmly wedge in place. Matrices are flexible to allow contouring to provide proper contact and anatomy.

Matrices can be used side-by-side to restore adjacent teeth.



3

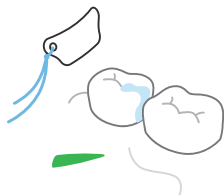

Perform procedure, remove matrix and dispose. Dental floss tied to the straight matrices allows for easy removal.

**Have additional questions? Need to re-order? Call us at:  
1.800.854.7949**

**FREE Ground Shipping on Qualified Orders**

*Continental US only, certain restrictions apply, see website for full details*

 [denovodental.com](http://denovodental.com)



Denovo Dental Inc.  
5130 Commerce Dr.  
Baldwin Park, CA, USA

**EC REP**

KinderDent, Gutenbergstrasse 7  
D - 28844 Weyhe, Germany



Surgical grade stainless steel, which contains nickel (9.58%). Nickel has been shown to cause allergic reactions in some patients.



Do not reuse. Appliances are intended for single use only. If appliance is only used for a trial fit, sterilize for future use.



Aspiration/Ingestion possibility.

Denovo products are sold clean but not sterile; sterilization prior to use is recommended for patients with compromised immune systems.

REV 121